



PYLE

PyleUSA.com



PHLHA57

Digital Voice Amplifier

Rechargeable Invisible Earstud


Voice Sound Amplifier



USER MANUAL

KNOW YOUR DEVICE


fig.1

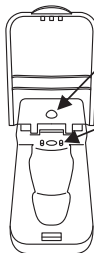

charging light indicator

charging connections

fig.2

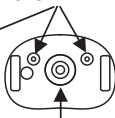

volume


adjustment button

fig.3




ear
tip

fig.4

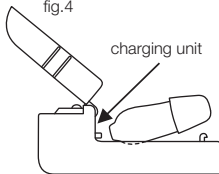

charging unit


fig.5

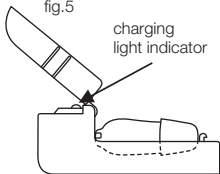

charging
light indicator

fig.6

Micro USB slot

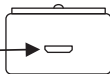

CHARGING INSTRUCTIONS

Charge your Digital Voice Amplifier

1. Ensure that the **Digital Voice Amplifier** is **OFF**.
The **Digital Voice Amplifier** will not charge if the device is **ON**.
2. Slide the **Digital Voice Amplifier** into the charging unit as shown in fig.4 and fig.5, with the charging connections, fig.2, facing the charging light indicator, fig.5. The small arrow at the base of the **Digital Voice Amplifier** should point towards the charging light indicator.
3. Push lightly on the **Digital Voice Amplifier** until it clicks into place.
4. Insert charging cable to the slot, fig.6, and the other end into the USB socket of your AC Adaptor. Switch ON at the mains and the charging light indicator will light up. The cable can be used with most standard USB AC adaptors.
5. If the charging light indicator does not light up check that the USB cable has been inserted correctly and the mains power is switched ON.
6. The charging light indicator is red when charging. If the charging light indicator shows red and green together the **Digital Voice Amplifier** has been inserted incorrectly, remove and re-insert the **Digital Voice Amplifier**. Once charged the charging light indicator will turn green.
7. A full charge takes a minimum of 2 hours, and will

allow for up to 12-20 hours of usage, depending on sound output.

Please note that the rechargeable battery inside the **PHLHA57** should not be removed and/or replaced. Attempting to self-service will void the warranty.

- The rechargeable battery is designed to last for at least 400 recharge cycles. With proper care, the battery can last more than 800 charge cycles.
- To extend the life of the battery, keep the **PHLHA57** in the charging unit and on charge when not in use.

OPERATING INSTRUCTIONS

Before using your **PHLHA57**, ensure that it is fully charged.

• TURNING ON AND OFF

To turn ON or OFF press the volume adjustment button, fig.2

1. Gently place the **Digital Voice Amplifier** into the ear, once in place increase the volume by pressing the volume adjustment button.
2. Using your finger, keep pressing the volume adjustment button until you reach a comfortable level. There are six levels of volume, you will hear a beep after each level and a double beep when you have reached maximum volume.

3. Once you have set the volume level the **Digital Voice Amplifier** will remember this setting on your next usage.
4. Always turn OFF before inserting into the charging unit. Always keep the **Digital Voice Amplifier** in the charging unit and ON charge when not in use.

CARE & CLEANING

- Before and after each use remove all debris from the ear cap.
- Ensure the ear cap hole is free of any debris. Use the cleaning brush to clean any earwax adhered to the **Digital Voice Amplifier**

• EAR CAP REPLACEMENT

If the Ear Cap can no longer be properly cleaned, or if they have any tears or damage, then it is time to replace the Ear Cap.

1. Remove the old Ear Cap



2. Turn up the rim of the new Ear Cap



3. Insert new Ear Cap on **Digital Voice Amplifier**



4. Turn the rim of the new Ear Cap down



• **Troubleshooting**

THE DIGITAL VOICE AMPLIFIER WILL NOT CHARGE:

1. Is the **Digital Voice Amplifier** seated into the charging unit correctly.
2. Is the **Digital Voice Amplifier** turned OFF? Ensure that it has been turned OFF as it will not charge if left ON.
3. Is the USB cable inserted correctly at both ends in both the charging unit and the AC Adaptor, and is the mains power turned ON?

CAN'T HEAR ANYTHING:

1. Has the **Digital Voice Amplifier** been turned ON? Check that the volume has been turned up by pressing the volume adjustment button.
2. Is the ear tip blocked with wax or debris? Clean the ear tip using the cleaning brush. Do not wash the ear tip in water or use any cleaning solutions.
3. Is the **Digital Voice Amplifier** fully recharged? Place and ensure the **Digital Voice Amplifier** in the charging case for 1-2 hours.

WARNING:

This product is not intended for use by anyone under the age of 18. It is designed to rectify mild to moderate hearing impairment. You may incur interference when using a mobile phone or radio near the device. Remove your **Digital Voice Amplifier** when showering, washing your face, using hair spray, etc. Remove prior to vigorous exercise and when sleeping.

PLEASE NOTE:

Medical studies show that your brain requires up to 21 days to adjust to a new hearing device. Your brain will need to re-learn how to process sounds by prioritizing foreground over background sounds, this will subside after few weeks. We recommend wearing PHLHA57 for 30 to 60 minutes a day at first and gradually extend the time that you wear it. Medical studies have shown that you will get far greater satisfaction and the BEST results by wearing two devices.

What's in the Box:

- Voice Amplifier
- (4) Ear Tip
- Charging Base
- Manual

PHLHA57

DIGITAL VOICE AMPLIFIER

Features:

- Digital Voice Amplifier
- Universal Fit Earstud
- Built-in Rechargeable Lithium Battery
- Interchangeable Ear Tip to Seal Out the Noise
- Selectable Listening Modes: Normal, Noisy Background
- Unique Ergonomic Design to Prevent Occlusion Effect
- Circular Volume Adjustment Knob
- Active Noise Reduction
- Over Discharge/Charge Feature
- Wider Dynamic Range Compression

Technical Specs:

- Construction Material: ABS
- Input Noise: $\leq 30\text{dB}$
- Sensitivity: $\geq 30\text{dB}$
- THD: 4%
- Max Sound Output: $120\pm 5\text{dB}$
- Frequency Range: 200-5000Hz
- Power Supply: D.C. 3.7-5V
- Working Voltage: D.C. 1.5V
- Working Current: $\leq 4\text{mA}$
- Built-in Rechargeable Battery: Lithium-ion
- Dimensions (L x W x H): 0.94" x 0.55" x 0.35" -inches
- Sold as: 1